
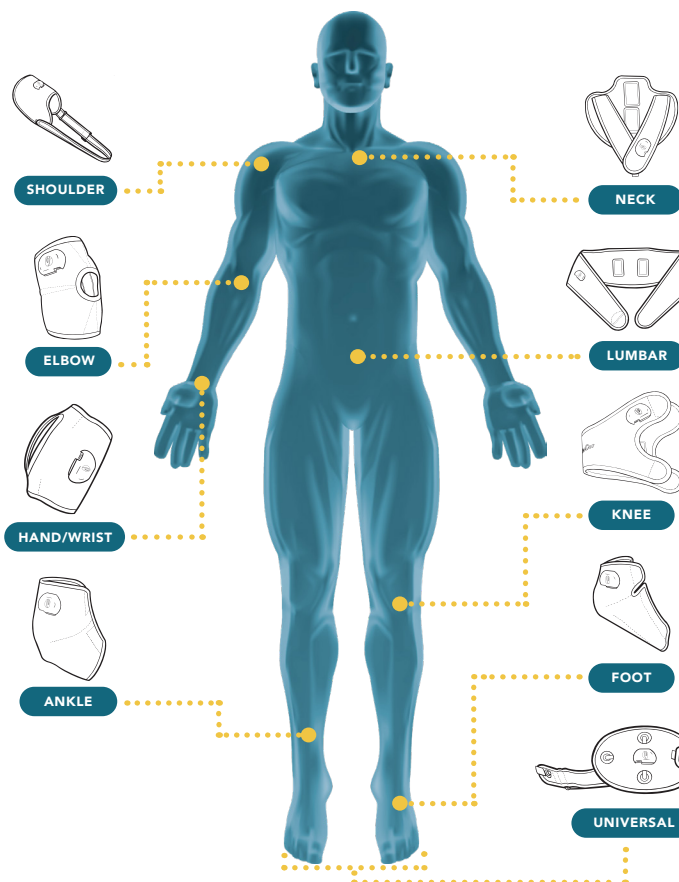
ACTIVE SYSTEM

QUICK START GUIDE

QUICK START GUIDE

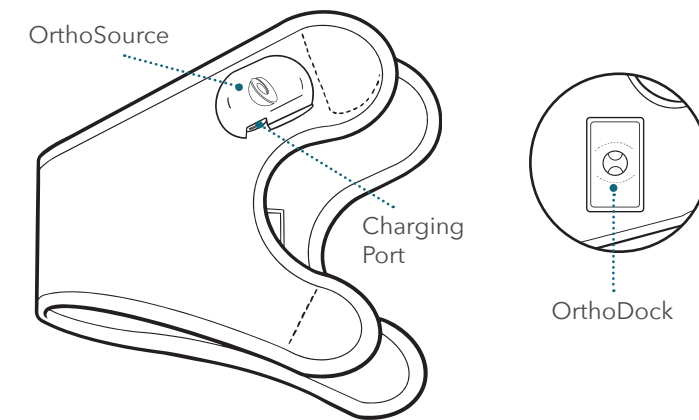PACKAGE COMPONENTS AND ACCESSORIES

DEFINITIONS:

OrthoSource: Module containing electronics necessary to deliver Pulsed Electromagnetic Field (PEMF) technology

OrthoDock: Location where OrthoPods are inserted on the Active System

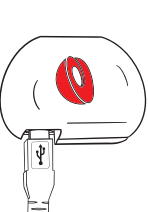
Active System: Multiple-use device designed to house both the OrthoSource and OrthoPods when in use and direct therapy to the appropriate areas of the body.

AC Adapter/Charger: Micro-USB compatible AC charger used to charge the OrthoSource. 3.7 V DC internally powered.

These therapy accessories are provided separately:

OrthoPods: Single-use therapy pods

PRIOR TO USE: CHARGING

- Make sure to fully charge your device prior to the first time you use it.
1. Plug the power cord into the micro-USB port on the bottom of the OrthoSource. A red light will indicate the device is charging.
 2. When the device is fully charged, the red light will turn off. The device takes 90 minutes to charge.

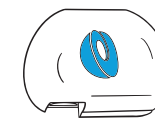
QUICK START GUIDE

STEP 1: PREPARE PODS

1. Open a new package of OrthoPods 5-10 minutes before you start a therapy session to allow them to warm up.
2. Follow the operating instructions included with the OrthoPods.

STEP 2: INITIATE THERAPY


1. Insert the OrthoPods into the OrthoDocks on the OrthoCor Active System. The magnetic fasteners should ensure a strong connection.
2. A blue light will appear on the OrthoSource, indicating that the therapy session has begun.
3. Therapy will run for 2 hours.

STEP 3: POSITIONING

Find your device and follow the instructions. Adjust to fit comfortably but do not overtighten.

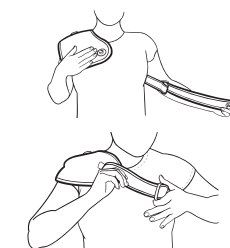
SHOULDER

1. Determine which shoulder needs to be treated. The strap first should be secured to the magnetic closure on the back of the shoulder - this should be the closure farthest away.
2. Place the device on the shoulder, grab the strap from behind the body and pull the strap across the chest to the magnetic closure on the front of the device. Adjust the strap using the buckle until it's comfortable and holds the device in place on the shoulder.
3. To remove the device, pull on the release tab on the front side of the device and pull up away from the shoulder until the magnetic closure releases.


QUICK START GUIDE

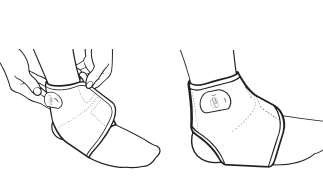ANKLE


1. Place the heel of your foot into the hole located in the center of the ankle device, ensuring the OrthoSource is above the ankle.
2. Wrap the device around the front of the ankle. The OrthoPods should be on each side of the ankle. Adjust the device to fit comfortably, but do not overtighten.

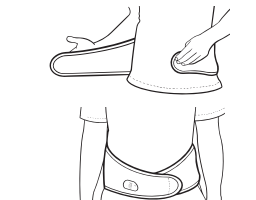BACK

1. Position the back device around the waist, use the tabs at the end of the straps to help position the device. The OrthoPods should be positioned directly over the intended treatment area.
2. Fasten the device comfortably around the front of the body. Adjust to fit, but do not overtighten.

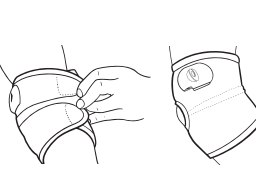ELBOW

1. Position the elbow device by placing the elbow joint in the hole in the center of the device. The OrthoSource should be positioned just above the elbow.
2. The OrthoPods should be positioned on each side of the elbow joint. Adjust the device to fit comfortably, but do not overtighten.

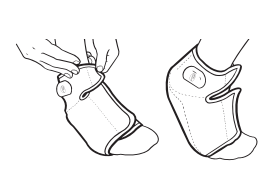
KNEE

1. Position the knee system with the logo centered on the back of the knee. Wrap the tabs around to the front of the knee.
2. The OrthoSource should be positioned just above the knee. OrthoPods should be positioned on each side of the knee joint.
3. Fasten the tabs comfortably above and below the knee. Adjust to fit but do not overtighten.


QUICK START GUIDE

FOOT


1. Place the heel of your foot into the hole located in the center of the foot device ensuring the OrthoSource is above your ankle. Wrap the device around the foot.
2. Fasten the device on the front of the foot. Adjust to fit, but don't overtighten. The OrthoPods should be positioned directly below the foot.
3. While wearing the device do not put weight or walk on the OrthoPods.

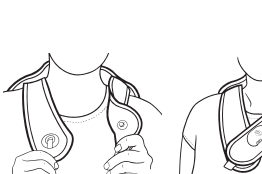NECK

1. Position the neck device with the OrthoPods against the back of neck. Grab the two front straps and pull forward and bring together until the magnetic closure automatically snaps closed. Make sure the OrthoPods are in contact with the treatment area after closure.
2. To remove the device, pull on the release tab on the top strap until the magnetic closure releases and then remove the device.

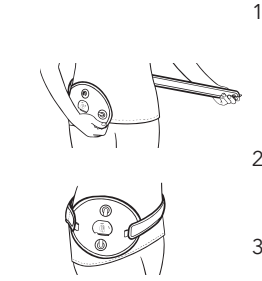
UNIVERSAL

1. Place the end of the strap onto one of the fasteners near the OrthoSource until the magnetic fastener engages. Place the device over the treatment area. Wrap the strap around the treatment area and adjust the strap length for comfort.
2. Secure the end of the strap to the magnetic fastener on the opposite side of the device. All four fasteners may be used interchangeably to secure the device.
3. To remove the device, pull on the release tab on the top strap until the magnetic closure releases and then remove the device.


QUICK START GUIDE

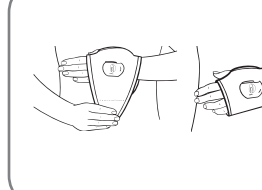WRIST

1. Place thumb through the thumb slot, ensuring the OrthoSource is sitting on top of the hand or wrist. Wrap the end of the wrist/hand device around the hand or wrist and fasten.
2. Slide the device up or down the hand or wrist to ensure it is covering the desired treatment site.

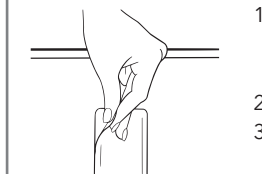
STEP 4: ENDING THERAPY

1. After 2 hours, the blue indicator light on the OrthoSource will flash for 2 minutes before turning off – indicating the end of the therapy.
2. Remove the Active System from the body.
3. Firmly grasp each OrthoPod and pull it away from the device until the connection with the OrthoDock is released. Dispose of the used OrthoPods in the trash.

RECHARGING INFORMATION

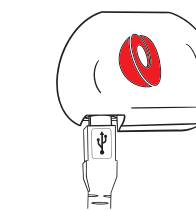- A freshly charged Active System should have sufficient power for two full therapies before needing to recharge.
- Leaving a fully charged device connected to the charger will not harm the battery; the device may be left charging until the beginning of the next therapy.

To learn more about setting up and using your device, visit us at: www.orthocormedical.com.

QUICK START GUIDE

FREQUENTLY ASKED QUESTIONS

How do I know when my device is charged?

When your device is fully charged, the red light will turn off.

What if the red light does not come on when I'm charging the device?

The red light will only turn on while the battery is charging. Make sure there is power to the outlet you are using, and the micro-USB charger is fully inserted into the OrthoSource charging port. If the red light still will not turn on, please contact OrthoCor Medical.

What if the blue light does not turn on when I insert new OrthoPods?

Make sure the OrthoCor device is fully charged, and the OrthoPods are properly seated in the OrthoDock. The blue light should appear to indicate the therapy session has begun. If the issue persists, please contact OrthoCor Medical.

How do I know that the therapy is completed?

After 2 hours, the blue indicator light on the OrthoSource will flash for 2 minutes indicating the end of the therapy. The light will completely turn off after 2 minutes.

What is the warranty policy?


OrthoCor Medical has a six-month manufacturer's warranty on any defects. OrthoCor Medical does not accept returns or offer refunds, except for cases that fall within the six-month warranty policy.

LED INDICATION

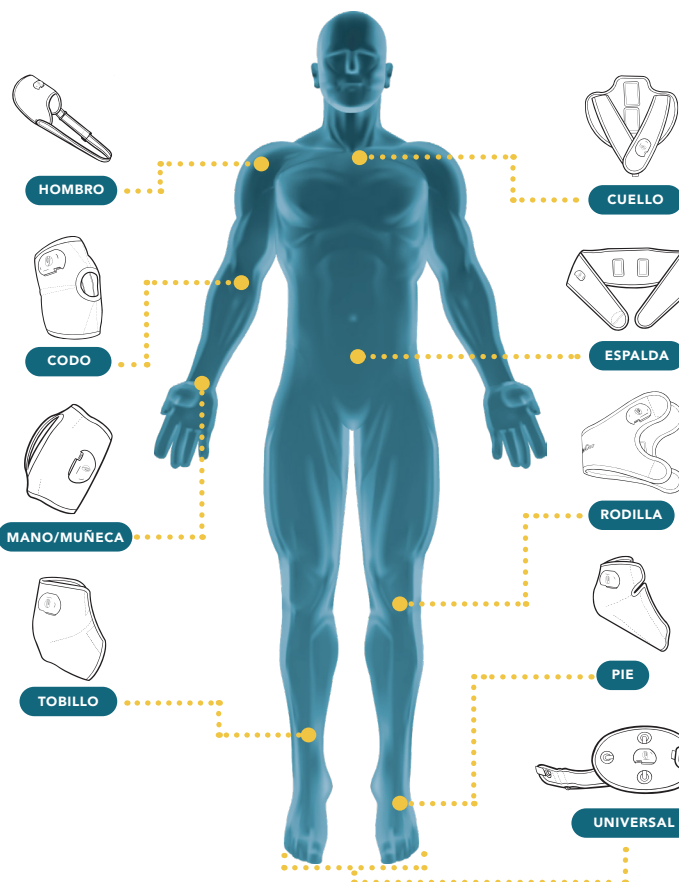Battery Status	LED Indication
Charging	Red
Active Therapy	Blue
Therapy Ending	Flashing Blue

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